


LifeStream™ 5.8

Web Interface

What's New?

We've updated LifeStream to make it faster, simpler, and more efficient for you to use - while enhancing the experience it delivers to your patients, caregivers, and clinicians.

More information about our complete release is available from the *Help* menu of LifeStream.

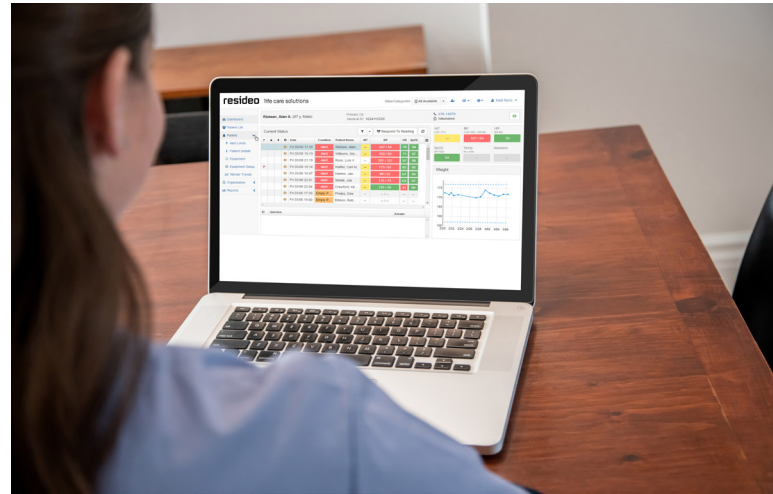

LifeStream is simply faster, simply better. Now more than ever.

- Preset configuration templates standardize and simplify the enrollment process, while maintaining flexibility to meet specific needs.
- EMR interface enhancements make entering notes faster and more efficient by capturing the most important information, while eliminating duplication.
- A revised hospital admission report improves workflow and efficiency.
- Streamlined Genesis Touch® configuration updates speed patient use of new schedules by updating the tablet within minutes.
- An increased number of video visit participants expands flexibility to involve more family and friends, care providers, insurers, or others by supporting up to ten people on a video visit.

LifeStream makes support more efficient, information more secure.

- Expanded Genesis Touch remote control permits LifeStream users to unlock and reboot a patient's tablet (Genesis Touch 3.10 or later), adjust advanced settings, and troubleshoot issues in less time.
- A custom unlock code and other platform updates increase information security while improving clinical efficiency.

To learn what else is new, browse the *Help* menu from LifeStream.

Technical Requirements

SOFTWARE REQUIREMENTS

Web Browser and Operating Systems:

Microsoft Internet Explorer® upgraded to Version 11 or the **Microsoft Edge Web browser**, with all system updates.

Google Chrome™ for Microsoft or Mac upgraded to the most recent stable version.

Compatibility with other Web browsers has not been validated. Use of other web browsers may result in difficult-to-access controls or a cluttered display that may adversely impact ease of use.

LifeStream's Web interface has been validated with a desktop Web browser, on the Operating Systems specified above. Install and maintain all Operating System updates on any computers interacting with LifeStream™ to protect patient data and your system.

Portable Document Format (PDF) Reader:

Use Adobe® Acrobat® Reader, or an updated PDF reader, to view files.

Anti-Virus Software:

Install and maintain current anti-virus software on computers interacting with LifeStream™ to help protect patient data and your system.

SYSTEM REQUIREMENTS

Display: A monitor screen capable of supporting at least a 1200 x 768 resolution.

Peripherals: Mouse and keyboard.

Internet: An Internet connection is required.

SYSTEM RECOMMENDATIONS

Processor: 1GHz or faster x86- or x64-bit

Memory: 1GB RAM 32-bit; 2GB RAM 64-bit

System Time: This web interface displays times in the time zone of the user's Web browser. To check the correct time for your time zone, use: <http://www.time.gov>.

For more information

lifecaresolutions.resideo.com

Life Care Solutions

3400 Intertech Drive, Suite 200
Brookfield, WI 53045 USA

© 2021 Resideo Technologies, Inc.

LifeStream™ Updates: LifeStream™ uses a platform approach that limits the likelihood that incremental updates to either a supported operating system or browser would impact the product. However, if a deleterious impact were reported, LifeStream™ supports an update mechanism that creates a path to deliver revised software. These impacts are limited to the specific client workstation and do not impact the retrospective collection of patient data.

Video Visits: A license is required to host video visits with a patient. All visits must include at least one patient. Genesis Touch® patient monitors must be upgraded to a compatible version, connected to Wi-Fi®, or licensed with a video-over-cellular data plan. Participants and hosts using Internet Explorer will be asked to download and install a Vidyo.io™ plugin, or a Vidyo.io™ application.

Video Visits may not be used on the Microsoft Edge Web browser.

Workstation: Any location where a user will access and interact with LifeStream™. Each workstation, both within the agency and those at any remote location, should be evaluated by your IT support staff to ensure the specifications have been met. The LifeStream™ Web interface may be accessed without an installation on a workstation, via a web interface.

Internet Access: The LifeStream™ Web client uses https:// connections. If an exception needs to be made in your Proxy Server or firewall to allow access, we recommend it be made to the domain vitalsmanager.com. Some network proxy configurations require authentication be entered at the web browser level, and may require that an exception be entered in the proxy for the system to work.

Some network proxy configurations require authentication be entered at the web browser level, and may require that an exception be entered in the proxy for the system to work.

HIPAA Medical Privacy Standards: LifeStream™ has been designed to comply with HIPAA regulations. It is the organization's responsibility for complying with privacy and security statutes for full HIPAA compliance, including any necessary and related policies and procedures. All users of the system hold responsibility and accountability for protecting the privacy of a client's health information.

Credentials: To install or access LifeStream™, authorized users need a username and password. To access the LifeStream™ Web interface, you may login at the web address given to you in your welcome documentation, or by your administrator.

Monitor Compatibility: All warranted Life Care Solutions monitors and their compatible peripherals, including Genesis Touch and Genesis DM monitors.

HomMed®, LifeStream™ and Genesis Touch® are either trademarks or registered trademarks of Resideo Technologies, Inc. All other product names, trademarks, logos and brands cited herein, are property of their respective owners.

Android™, Google, Chrome™ browser and the Google logo is a trademark of Google Inc.; Adobe® and Adobe® Acrobat® Reader are registered trademarks of Adobe Systems Incorporated; Microsoft, Windows, Excel, Internet Explorer and Windows 2000 are either registered trademarks or trademarks of Microsoft Corporation. Wi-Fi is a registered trademark of Wi-Fi® Alliance. VIDYO, the Vidyo cube icon, the Vidyo Partner Logos and names of Vidyo Inc.'s products and offerings are trademarks or registered trademarks of Vidyo, Inc., in the United States and in other jurisdictions. CPT® is a registered trademark of the American Medical Association.

Customer Service: 1-888-353-5404 | Ext. 1

Resideo Life Care Solutions is not responsible for the technical support of any workstations defined herein.

E21-0011 | P2000EN.16B | Feb 2021